



Automatic Microneedling System for Professionals

Collagen Induction Therapy & Drug, Cosmetic Delivery System



WOORHI MECHATRONICS CO., LTD.

5th Floor, Woorhi Mecha building, 58, Sanbon-ro, 48beon-gil, Gunpo-si, Gyeonggi-do, 15846, Republic of Korea
Tel +82. 31. 457. 5032 | E-mail sales@woorhimecha.com

CODE-X

Raffine
Automatic Micro needle Therapy System

Microneedling for Professional

Automatic Microneedling Therapy System

Minimally invasive procedure for skin makes small puncture on the skin layer.
Skin's self healing process produces collagen and elastin which help regeneration.
Also a lot of active ingredients of cosmetics are absorbed into a deep layer of skin.

CODE-X Innovative Wireless Microneedling Device

- Wide range of speed from 3,500RPM up to 5,500RPM by 5 steps
- High capacity of battery for 3hours using time
- Battery management system for maintaining a stable power and speed at low battery status
- Specific design for reliable drag-free
- Life time warranty



Microneedle Cartridge

- 9Pins, 12Pins, Nano needle
- Anti-backflow functions



Raffine Medical Microneedling Device

- Wide range of speed from 4,500RPM up to 6,500RPM
- Medical CE, MFDS certified
- Plug-in system for full power treatment and consistent output
- High quality motor that guarantees smooth skin needling even with the deepest
- Life time warranty



PROTOCOL

STEP 1. CLEANSING

- Clean the treatment area (avoid deep and chemical cleansing type)

STEP 2. APPLY THE SOLUTIONS(AMPOULES, SKIN-BOOSTER...)

- If the treatment has a medical purpose with using over 0.5mm length, recommend to apply the numbing first.
- Apply the cosmetic or medicine solution to be absorbed into the skin layer.

STEP 3. MICRONEEDLING TREATMENT

- Length of needle and sessions depends on the treatment purpose.

STEP 4. POST CARE

- Sunscreen care is required. Avoid using exfoliating, regenerating and moisturizing products for at least 3 days.